




## Veloped Cane holder – Assembly Instruction

2-piece cane holder for the Veloped that also works as a safety strap when the Veloped is folded for transport or storage. The cane holder fits all Veloped models.

Scan the QR-code with your smart phone or device to view video instructions.

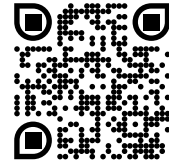

### ① Attach the rear holder.

Wrap the rear holder around the upper part of the rear vertical frame tube, just below the height-adjustment quick-release.



### ② Pull the strap tight.

Wrap the strap around the tube and attach it with the Velcro. Let the excessive length of the strap hang freely.



### ③ Adjust the rear loop.

The size of the rear loop can be adjusted by loosening the Velcro closure. You may need to gently squeeze the loop to get a rounded/circular shape.



#### ④ Attach the front holder.

Wrap the front holder around the upper frame tube, just behind the basket rail attachment. Make sure that the anti-slip patch is facing the frame tube.



#### ⑤ Adjust the front loop.

The size of the front loop can be adjusted by loosening the Velcro closure. You may need to gently squeeze the loop to get a rounded/circular shape.



#### ⑥ Attach the walking stick.

Attach the walking stick by pushing it from the rear, first through the rear and then through the front holder.



#### ⑦ Secure the frame.

You can use the cane holder strap to secure the frame when it is folded, when transporting or storing your Veloped.