



Manual: Trionic Cable Lock



1 Title1

Locate the screw on the bracket.



2 Title2

Unscrew the screw with a 4 mm allen key.



3 Title3

Pull the end of the strap out of the bracket.



4 Title4

Wrap the strap around the Veloped rear vertical frame tube (on the Walker, the semi-vertical frame tube).



5 Title5

Insert the end of the strap into the bracket.




6 Title6

Tighten the screw/strap with the 4 mm allen key until the strap is tight.



7 Title7

Cut off the excessive length of the strap with a cable cutter or knife.

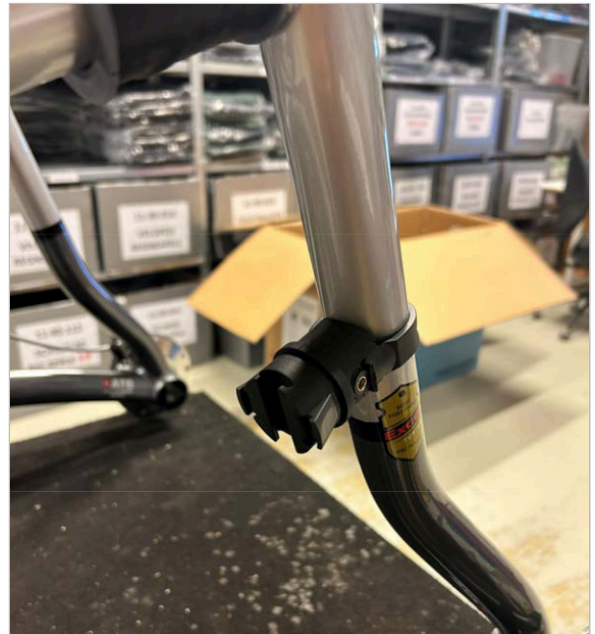


8 Title8

The photo shows the correct attachment of the bracket.



9 Title9

Attach the cable lock to the bracket from above, push the cable lock downwards until it clicks into the bracket.

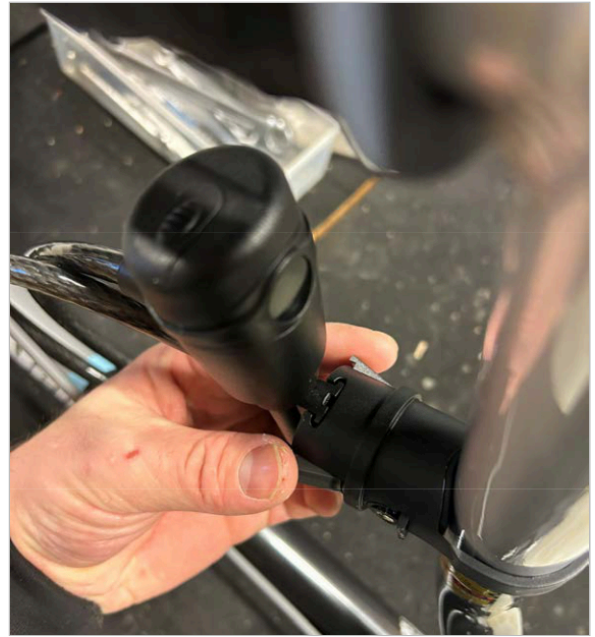

10 Title10

To detach the cable lock from the bracket, push the two grey buttons and remove the lock from the bracket.